

Instruction of use

Headlights by CLAR /
JANSEN



AS Medizintechnik GmbH • Sattlerstrasse 15, 78532 Tuttlingen, Germany • Tel 07461/966326 • Fax 07461/9663288

Headlights by CLAR / JANSEN: Art. Nr. 10-042-01 / 10-042-04

The instructions have to be well known and followed by every user. Misuse or any modification is not under the responsibility of the manufacturer.

Intended use:

Illumination of ear, nose and throat by medical well educated staff.

1. Assembling: please fix the mirror to the ball joint of the headband. The black headed screw have to be fastened so that the mirror still can be moved but does not move by itself.
2. Connection to power sources: before any connection is completed the power source has to be checked, if it is suitable (voltage, wattage, technical conditions). The connection to a power source is to be done by the banana plugs to a transformer (our codes 10-042-09) or a suitable battery box.

Warnings:

It is very important to check the power source before any connection. The necessary voltage is written on the label of the packing. All models need a current of 500 mA (= 0.5 A). **Never push the banana plugs directly into a mains voltage socket (110V/220V/240V)!**

Lamp exchange:

Unscrew the lamp carefully by turning to the left and **exchange it by a lamp of the same type.**

Attention:

The lamp can become very hot. So always after switching off wait a little while before touching the lamp. Protect patients and staff against touching the hot lamp. For proper and safe operating it is permitted only to use AS Medizintechnik GmbH lamps. Other lamps can cause damages of the headlight or the transformer. In result the CE sign will loose its validity.

Spare parts:

Item code.: 10-042-05 Lamp 6 V, 500 mA

Item code.: 10-042-95 Headband for 10-042-01

Item code.: 10-042-99 Spare mirror for 10-042-01

AS MEDIZINTECHNIK GMBH DOES NOT ACCEPT RESPONSIBILITY IF THIS CUSTOMER INFORMATION HAS BEEN VIOLATED PROVABLY.