

Assembly / disassembly instructions

Detachable Kerrison laminectomy punches



40-014-01 Z - 40-016-26 ZE
(Only valid for articles with „Z“)



AS Medizintechnik GmbH
Sattlerstraße 15
78532 Tuttlingen / Germany
Phone: +49 74 61 9663026
info@AS-Medizintechnik.de



To prepare for decontamination, the above-mentioned products must be opened and, if necessary, disassembled into their individual parts so that all surfaces are accessible to the disinfectant. The instruments must be disassembled as described and reassembled after cleaning:

For more information on reprocessing and handling, see the following documents:




Cleaning, Disinfection and Sterilization:
Instruction for use and handling:

[AU_100_EN_Master reprocessing instruction](#)
[GU_300_EN_Surgical instruments](#)

Disassembly


1. Close the handle.

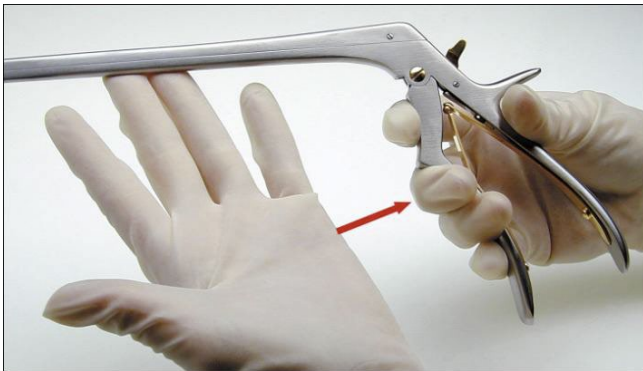


2. Press the golden latch downwards.



3. Open the handle.

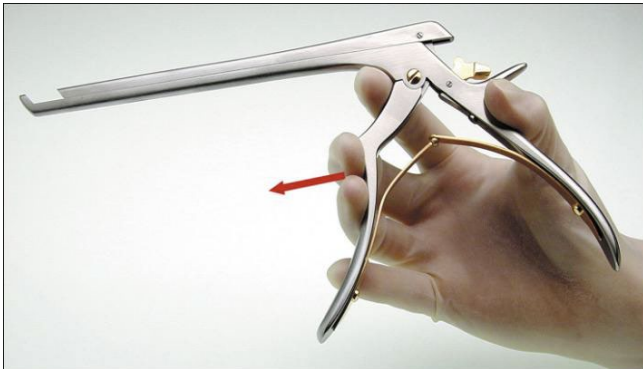

4. Press the back of the shaft to lift it upwards.
The instrument is now completely open.

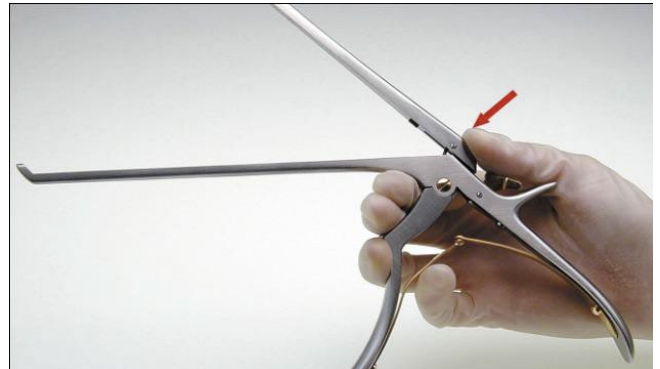

Assembly / disassembly instructions

Detachable Kerrison laminectomy punches



40-014-01 Z - 40-016-26 ZE
(Only valid for articles with „Z“)



AS Medizintechnik GmbH
Sattlerstraße 15
78532 Tuttlingen / Germany
Phone: +49 74 61 9663026
info@AS-Medizintechnik.de



To prepare for decontamination, the above-mentioned products must be opened and, if necessary, disassembled into their individual parts so that all surfaces are accessible to the disinfectant. The instruments must be disassembled as described and reassembled after cleaning:

For more information on reprocessing and handling, see the following documents:



Cleaning, Disinfection and Sterilization:
Instruction for use and handling:


[AU_100_EN_Master reprocessing instruction](#)
[GU_300_EN_Surgical instruments](#)

Assembly

1. Press the shaft parts into each other.



2. Close the handle.

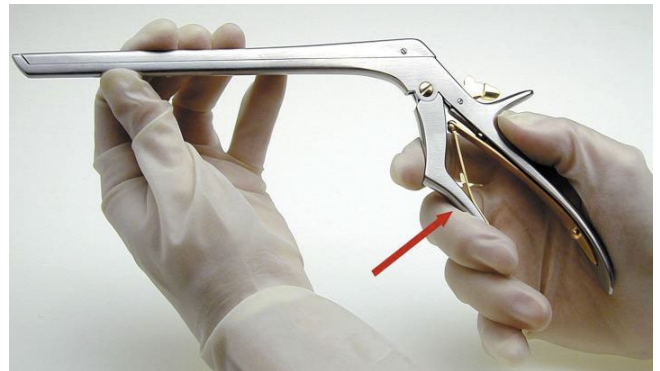

3. Press the golden latch upwards.



4. Open the handle. The Instrumnet is now completely assembled.

