

Assembly / disassembly instructions

Detachable laminectomy forceps with pistol handle



40-028-32 Z - 40-059-23 Z
(Only valid for articles with „Z“)



AS Medizintechnik GmbH
Sattlerstraße 15
78532 Tuttlingen / Germany
Phone: +49 74 61 9663026
info@AS-Medizintechnik.de



To prepare for decontamination, the above-mentioned products must be opened and, if necessary, disassembled into their individual parts so that all surfaces are accessible to the disinfectant. The instruments must be disassembled as described and reassembled after cleaning:

For more information on reprocessing and handling, see the following documents:




Cleaning, Disinfection and Sterilization:
Instruction for use and handling:


[AU_100_EN_Master reprocessing instruction](#)
[GU_300_EN_Surgical instruments](#)

Disassembly


1. Press button „A“ to release the mechanism.

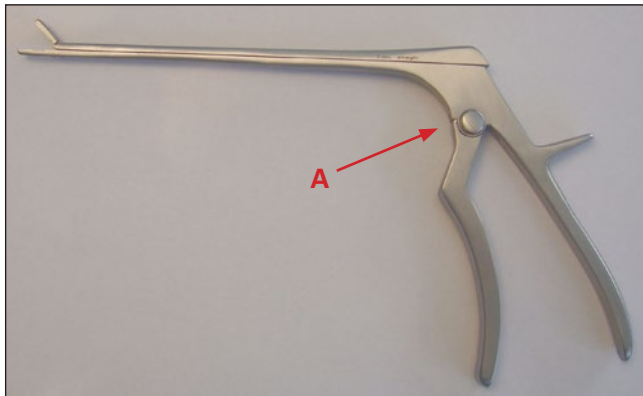

2. Pull the handle downwards.

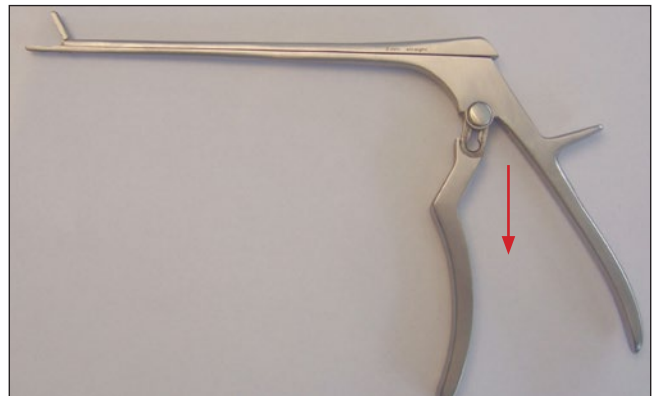

3. Fold the shaft upwards. The instrument is now completely open.

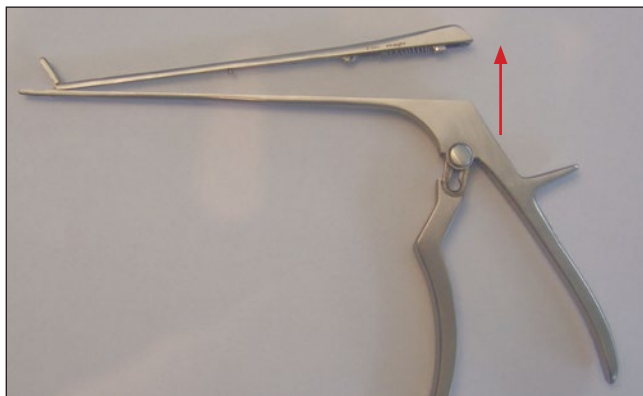

ATTENTION:

Do not overstretch the shaft or fold it up too far! (danger of breakage)

Assembly / disassembly instructions

Detachable laminectomy forceps with pistol handle



40-028-32 Z - 40-059-23 Z
(Only valid for articles with „Z“)



AS Medizintechnik GmbH
Sattlerstraße 15
78532 Tuttlingen / Germany
Phone: +49 74 61 9663026
info@AS-Medizintechnik.de



To prepare for decontamination, the above-mentioned products must be opened and, if necessary, disassembled into their individual parts so that all surfaces are accessible to the disinfectant. The instruments must be disassembled as described and reassembled after cleaning:

For more information on reprocessing and handling, see the following documents:



Cleaning, Disinfection and Sterilization:
Instruction for use and handling:


[AU_100_EN_Master reprocessing instruction](#)
[GU_300_EN_Surgical instruments](#)

Assembly

1. Fold the shaft down and slide the shaft parts into each other



2. Slide the handle upwards in the middle position.

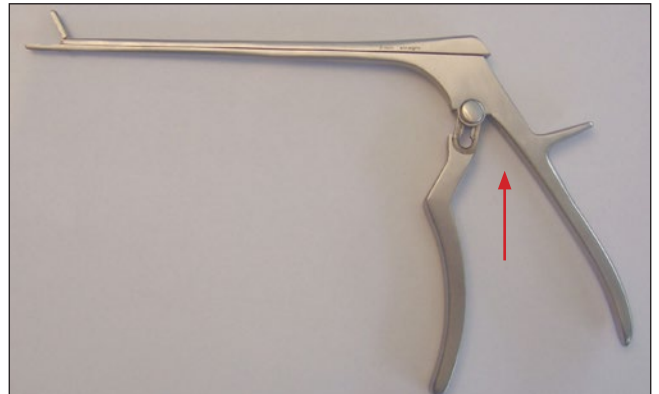

3. Press the handle until the button „A“ is engaged. The instrument is now completely assembled.

