

# Assembly / disassembly instructions

Detachable laminectomy forceps with ring handle



40-050-02 ZR - 40-069-14 ZRB  
(Only valid for articles with „ZR“)



AS Medizintechnik GmbH  
Sattlerstraße 15  
78532 Tuttlingen / Germany  
Phone: +49 74 61 9663026  
info@AS-Medizintechnik.de



**To prepare for decontamination, the above-mentioned products must be opened and, if necessary, disassembled into their individual parts so that all surfaces are accessible to the disinfectant. The instruments must be disassembled as described and reassembled after cleaning:**

For more information on reprocessing and handling, see the following documents:




Cleaning, Disinfection and Sterilization:  
Instruction for use and handling:


[AU\\_100\\_EN\\_Master reprocessing instruction](#)  
[GU\\_300\\_EN\\_Surgical instruments](#)

## Disassembly


1. Slide the latch „A“ to the left or right to the side.

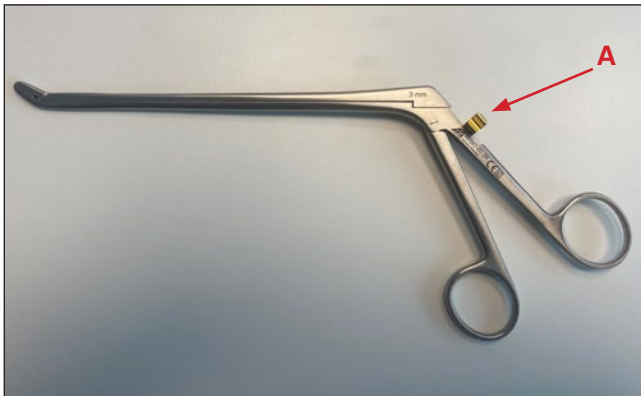

2. Push the handle forward.

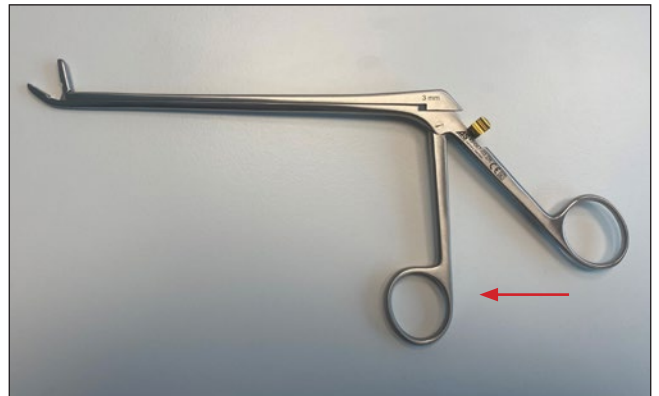

3. Fold the shaft upwards. The instrument is now completely open.

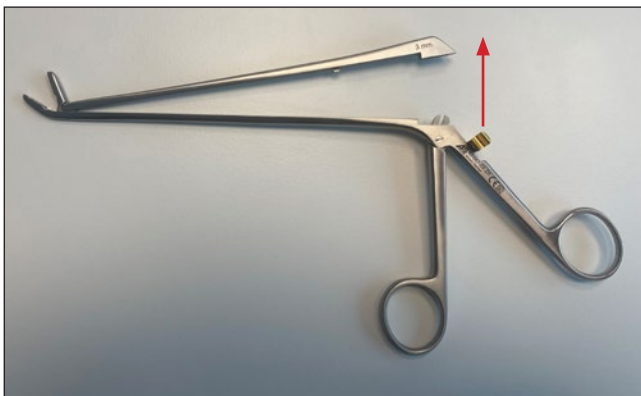

### ATTENTION:

**Do not overstretch the shaft or fold it up too far! (danger of breakage)**

# Assembly / disassembly instructions

Detachable laminectomy forceps with ring handle



40-050-02 ZR - 40-069-14 ZRB  
(Only valid for articles with „ZR“)



AS Medizintechnik GmbH  
Sattlerstraße 15  
78532 Tuttlingen / Germany  
Phone: +49 74 61 9 663026  
info@AS-Medizintechnik.de



**To prepare for decontamination, the above-mentioned products must be opened and, if necessary, disassembled into their individual parts so that all surfaces are accessible to the disinfectant. The instruments must be disassembled as described and reassembled after cleaning:**

For more information on reprocessing and handling, see the following documents:




Cleaning, Disinfection and Sterilization:  
Instruction for use and handling:


[AU\\_100\\_EN\\_Master reprocessing instruction](#)  
[GU\\_300\\_EN\\_Surgical instruments](#)

## Assembly

1. Fold the shaft down into the guide.

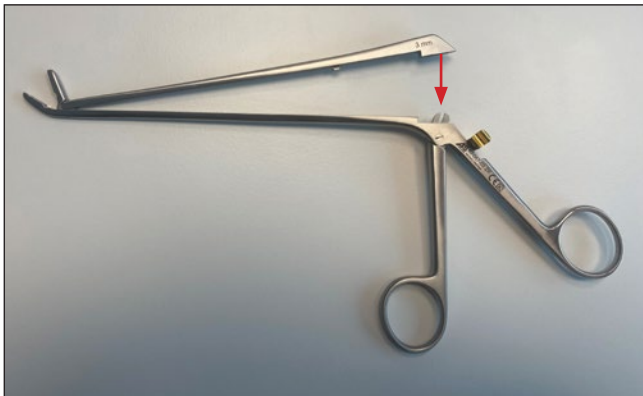

2. Push the handle back (close handle).

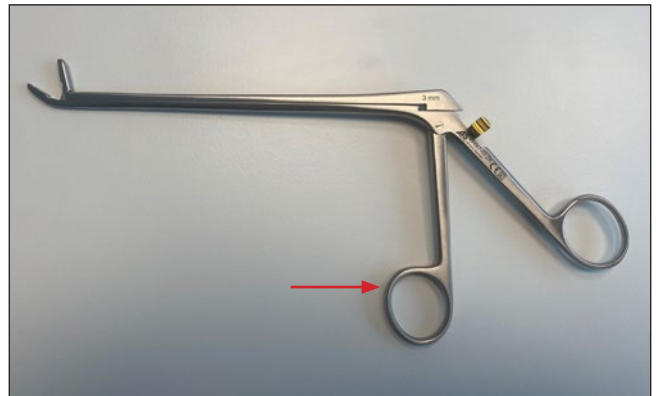

3. Slide the latch „A“ upwards.



4. The latch engages and fixes the shaft. The instrument is now completely assembled.

