

Assembly / disassembly instructions

Laminectomy spreaders with toothed rack



REF 20-302-00 - 20-305-00
20-350-00 - 20-370-33 T



AS Medizintechnik GmbH
Sattlerstraße 15
78532 Tuttlingen / Germany
Phone: +49 74 61 9663026
info@AS-Medizintechnik.de



To prepare for decontamination, the above-mentioned products must be opened and, if necessary, disassembled into their individual parts so that all surfaces are accessible to the disinfectant. The instruments must be disassembled as described and reassembled after cleaning:

For more information on reprocessing and handling, see the following documents:



Cleaning, Disinfection and Sterilization:
Instruction for use and handling:


[AU_100_EN_Master reprocessing instruction](#)
[GU_300_EN_Surgical instruments](#)

Disassembly

1. If necessary, remove the inserted valves / blades from the spreader. (These must be reprocessed separately)



2. Turn the right arm, over the toothed rack, until it can be removed at the end.

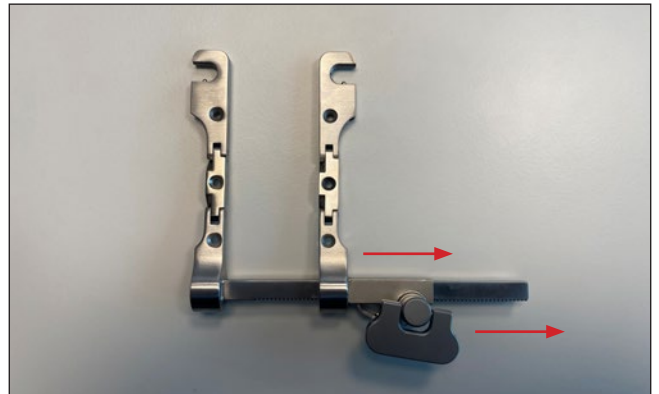

3. Remove the rotary levers „A“. The spreader is completely disassembled. (On certain spreaders, the rotary lever cannot be dismantled!)



Assembly / disassembly instructions

Laminectomy spreaders with toothed rack



REF 20-302-00 - 20-305-00
20-350-00 - 20-370-33 T



AS Medizintechnik GmbH
Sattlerstraße 15
78532 Tuttlingen / Germany
Phone: +49 74 61 9663026
info@AS-Medizintechnik.de



To prepare for decontamination, the above-mentioned products must be opened and, if necessary, disassembled into their individual parts so that all surfaces are accessible to the disinfectant. The instruments must be disassembled as described and reassembled after cleaning:

For more information on reprocessing and handling, see the following documents:



Cleaning, Disinfection and Sterilization:
Instruction for use and handling:


[AU_100_EN_Master reprocessing instruction](#)
[GU_300_EN_Surgical instruments](#)

Assembly

1. Insert the rotary lever „A“ (if not firmly connected) into the opening provided.



2. Put the right arm with the rotary lever on the toothed rack and turn it in to the left.

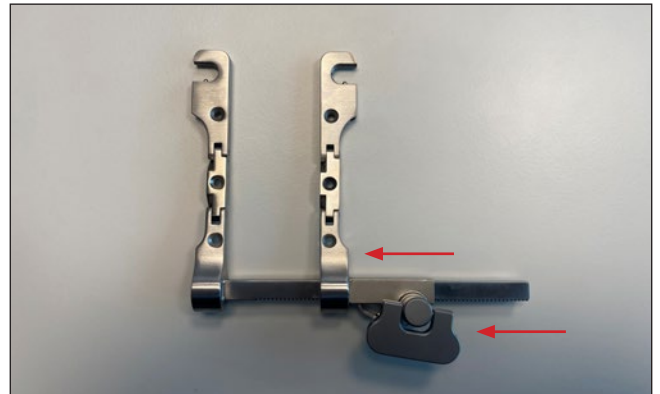

3. The spreader is completely assembled.

