

Assembly / disassembly instructions

Abdominal retractor (self holding)



REF 16-274-10 - 16-279-00



AS Medizintechnik GmbH
Sattlerstraße 15
78532 Tuttlingen / Germany
Phone: +49 74 61 9663026
info@AS-Medizintechnik.de



To prepare for decontamination, the above-mentioned products must be opened and, if necessary, disassembled into their individual parts so that all surfaces are accessible to the disinfectant. The instruments must be disassembled as described and reassembled after cleaning:

For more information on reprocessing and handling, see the following documents:




Cleaning, Disinfection and Sterilization:
Instruction for use and handling:


[AU_100_EN_Master reprocessing instruction](#)
[GU_300_EN_Surgical instruments](#)

Disassembly


1. Dismantle the nuts „A“, the retaining plate „B“ and the wing nut „C“ from the retractor.

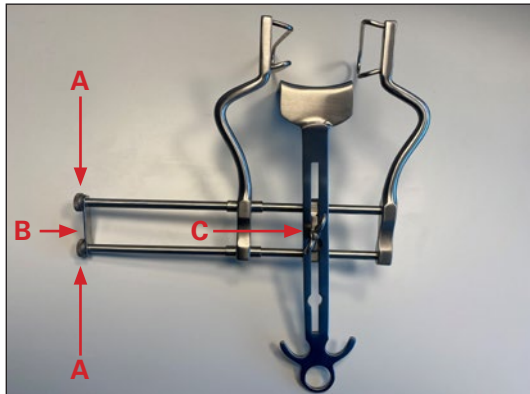


2. Remove the centre blade from the blade holder.

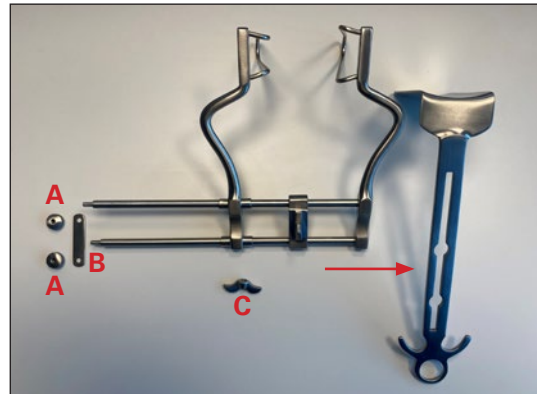

3. Slide the left-hand blade rod and the blade holder on the left out of the guide.

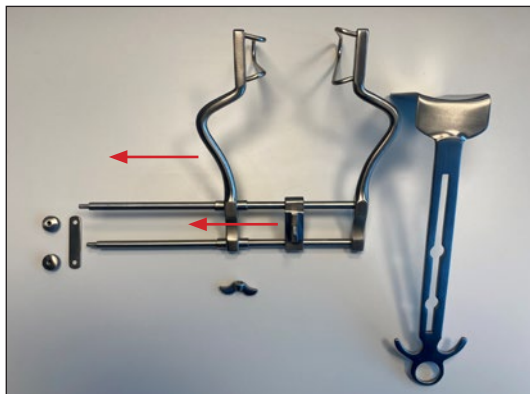

4. The retractor is completely disassembled.

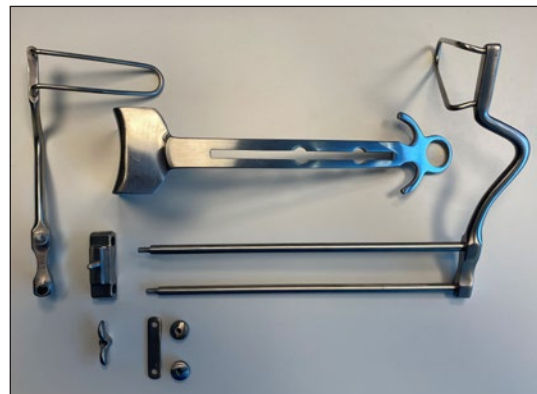

Assembly / disassembly instructions

Abdominal retractor (self holding)



REF 16-274-10 - 16-279-00



AS Medizintechnik GmbH
Sattlerstraße 15
78532 Tuttlingen / Germany
Phone: +49 74 61 9663026
info@AS-Medizintechnik.de



To prepare for decontamination, the above-mentioned products must be opened and, if necessary, disassembled into their individual parts so that all surfaces are accessible to the disinfectant. The instruments must be disassembled as described and reassembled after cleaning:

For more information on reprocessing and handling, see the following documents:




Cleaning, Disinfection and Sterilization:
Instruction for use and handling:


[AU_100_EN_Master reprocessing instruction](#)
[GU_300_EN_Surgical instruments](#)

Assembly


1. First push the blade holder and then the blade rod onto the retractor.

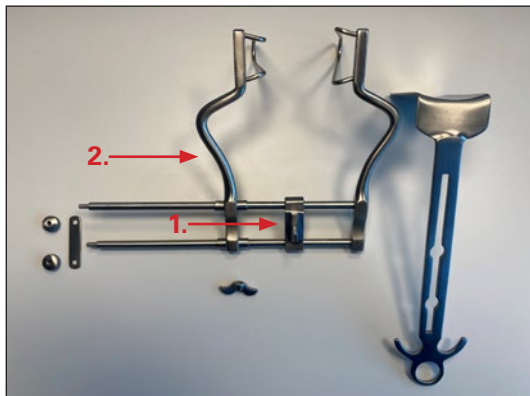

2. Place the centre blade in the blade holder and fix it with the wing nut „C“.

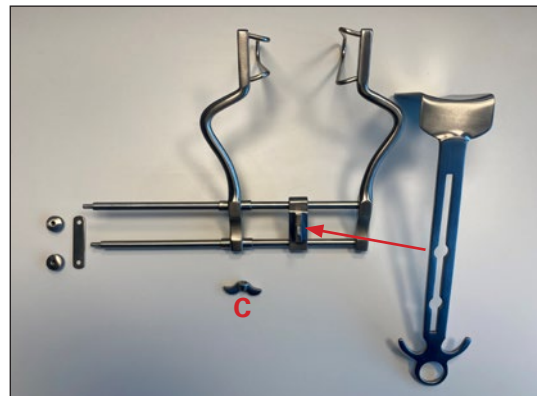

3. Slide the retaining plate „B“ onto the threaded rod and fix it with the nuts „A“.

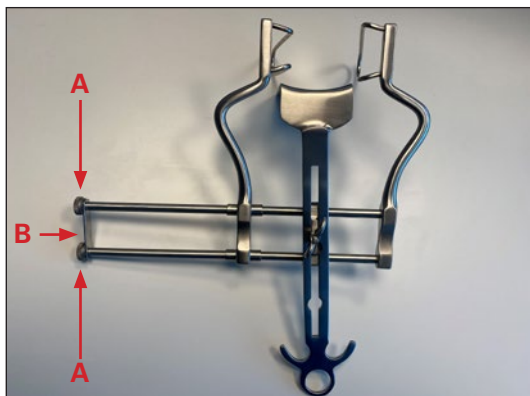

4. The retractor is completely assembled.

