

Assembly / disassembly instructions

Micro ear forceps, detachable



27-209-14 - 27-234-15
27-400-01 - 27-428-01



AS Medizintechnik GmbH
Sattlerstraße 15
78532 Tuttlingen / Germany
Phone: +49 74 61 9663026
info@AS-Medizintechnik.de



To prepare for decontamination, the above-mentioned products must be opened and, if necessary, disassembled into their individual parts so that all surfaces are accessible to the disinfectant. The instruments must be disassembled as described and reassembled after cleaning:

For more information on reprocessing and handling, see the following documents:




Cleaning, Disinfection and Sterilization:
Instruction for use and handling:


[AU_100_EN_Master reprocessing instruction](#)
[GU_300_EN_Surgical instruments](#)

Disassembly


1. Close the handle.

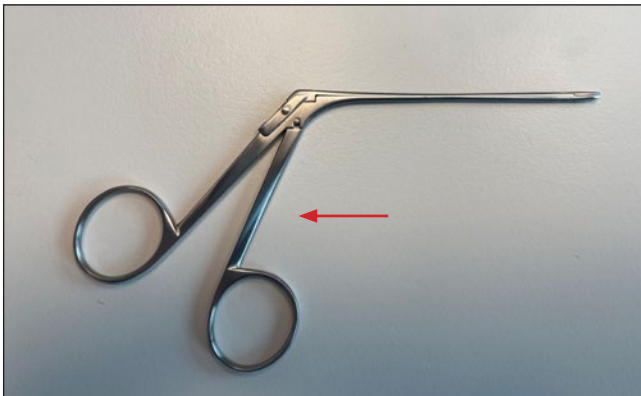


2. Slide the latch upwards

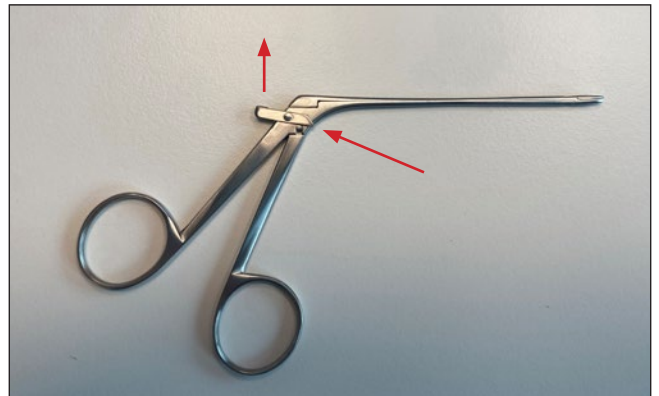

3. Turn the Instrumnet over and push the upper shaft away to the side with care.

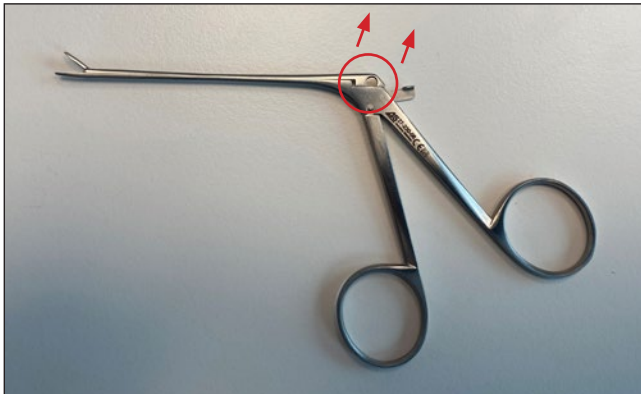

4. Carefully fold the shaft upwards. ATTENTION: Do not overstretch the guide in the middle.

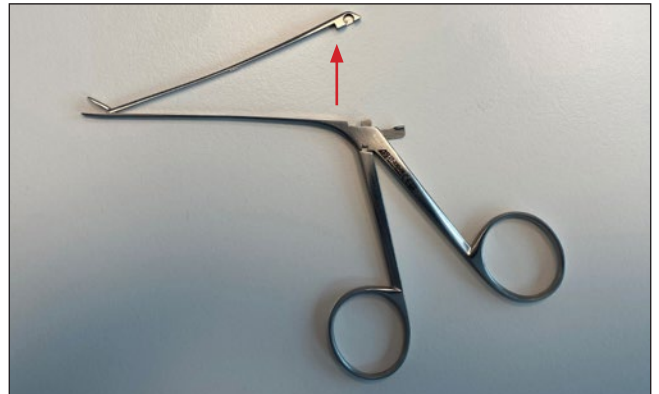

5. Pull the handle part away downwards. The instrument is completely disassembled.



ATTENTION:

Do not overstretch the shaft or fold it up too far! (danger of breakage)

ATTENTION:

Individual parts! Make sure that all parts are reassembled with the same components.

Assembly / disassembly instructions

Micro ear forceps, detachable



27-209-14 - 27-234-15
27-400-01 - 27-428-01



AS Medizintechnik GmbH
Sattlerstraße 15
78532 Tuttlingen / Germany
Phone: +49 74 61 9663026
info@AS-Medizintechnik.de



To prepare for decontamination, the above-mentioned products must be opened and, if necessary, disassembled into their individual parts so that all surfaces are accessible to the disinfectant. The instruments must be disassembled as described and reassembled after cleaning:

For more information on reprocessing and handling, see the following documents:




Cleaning, Disinfection and Sterilization:
Instruction for use and handling:


[AU_100_EN_Master reprocessing instruction](#)
[GU_300_EN_Surgical instruments](#)

Assembly


1. Push the handle part into the guide from below.

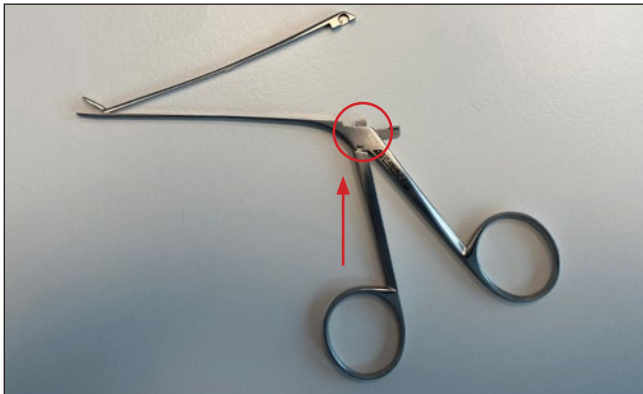

2. Fold the shaft down and carefully lever it into the round guide of the lower handle part.

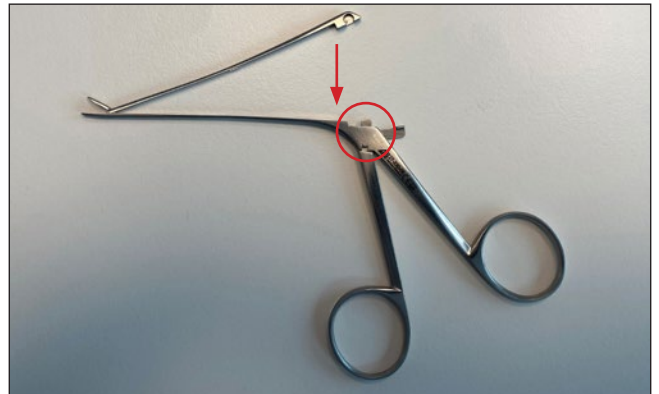

3. Make sure that the shaft already runs cleanly on the lower part in this setting.

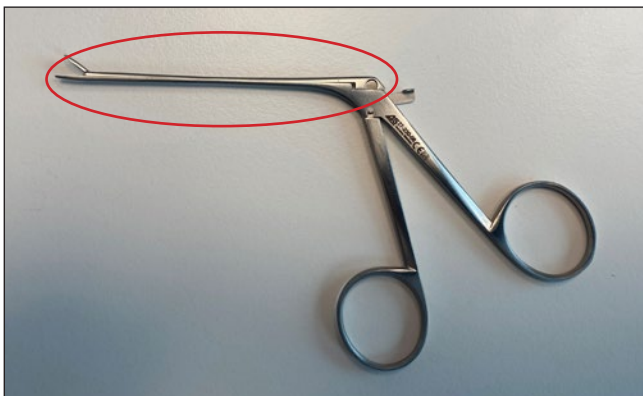

4. Turn the instrument over and slide the latch down. The instrument is now completely assembled.

