

Assembly / disassembly instructions

Forceps with sliding shaft and peacock, detachable



AS Medizintechnik GmbH
Sattlerstraße 15
78532 Tuttlingen / Germany
Phone: +49 74 61 9663026
info@AS-Medizintechnik.de



REF 26-411-30 - 26-420-50 / 27-460-25 - 27-461-48 / 27-600-10 - 27-600-12 / 27-650-01 - 27-650-14 / 33-362-28 - 33-362-42 / 35-410-28 - 35-421-23 / 36-080-19 - 36-082-12 / 40-036-30 A - 40-038-36 A (only valid for articles with „A“) / 40-038-00 - 40-038-13 / 40-040-05 - 40-043-01 / 40-072-07 - 40-076-35 / 40-090-14

To prepare for decontamination, the above-mentioned products must be opened and, if necessary, disassembled into their individual parts so that all surfaces are accessible to the disinfectant. The instruments must be disassembled as described and reassembled after cleaning:

For more information on reprocessing and handling, see the following documents:




Cleaning, Disinfection and Sterilization:
Instruction for use and handling:


[AU_100_EN_Master reprocessing instruction](#)
[GU_300_EN_Surgical instruments](#)

Disassembly


1. Slide the handle forward to the middle position.

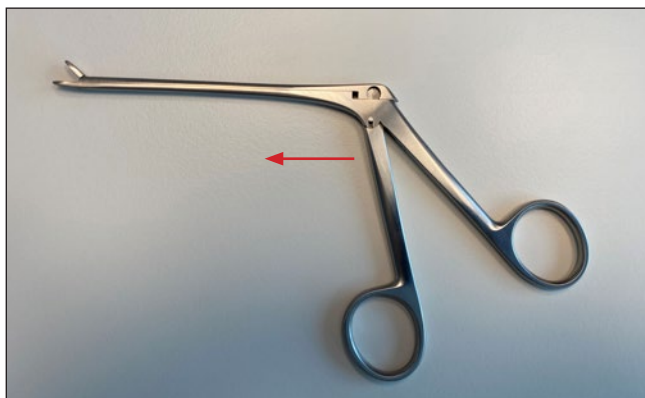

2. Lever the shaft backwards at the round opening and carefully fold the shaft upwards. ATTENTION: Do not overstretch the guide in the middle.

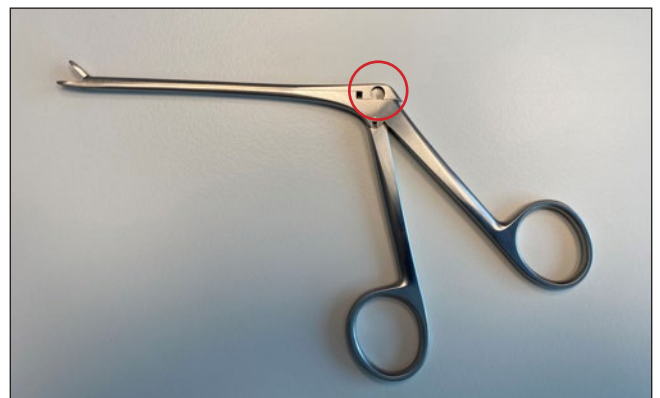

3. The shaft is folded upwards and the instrument is completely disassembled.

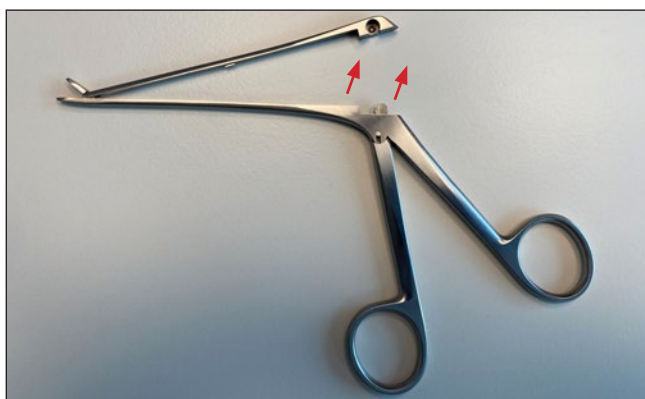

ATTENTION:

**Do not overstretch the shaft
or fold it up too far!
(danger of breakage)**

Assembly / disassembly instructions

Forceps with sliding shaft and peacock, detachable



AS Medizintechnik GmbH
Sattlerstraße 15
78532 Tuttlingen / Germany
Phone: +49 74 61 9663026
info@AS-Medizintechnik.de



REF 26-411-30 - 26-420-50 / 27-460-25 - 27-461-48 / 27-600-10 - 27-600-12 / 27-650-01 - 27-650-14 / 33-362-28 - 33-362-42 / 35-410-28 - 35-421-23 / 36-080-19 - 36-082-12 / 40-036-30 A - 40-038-36 A (only valid for articles with „A“) / 40-038-00 - 40-038-13 / 40-040-05 - 40-043-01 / 40-072-07 - 40-076-35 / 40-090-14

To prepare for decontamination, the above-mentioned products must be opened and, if necessary, disassembled into their individual parts so that all surfaces are accessible to the disinfectant. The instruments must be disassembled as described and reassembled after cleaning:

For more information on reprocessing and handling, see the following documents:




Cleaning, Disinfection and Sterilization:
Instruction for use and handling:


[AU_100_EN_Master reprocessing instruction](#)
[GU_300_EN_Surgical instruments](#)

Assembly

1. Slide the handle forward to the middle position.

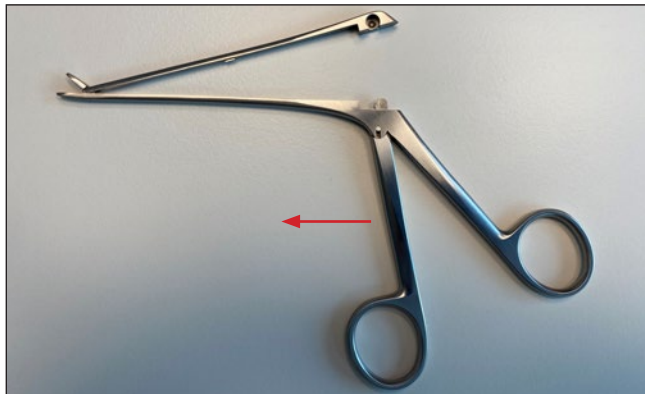

2. Fold the shaft down and carefully lever it into the round guide of the handle part.

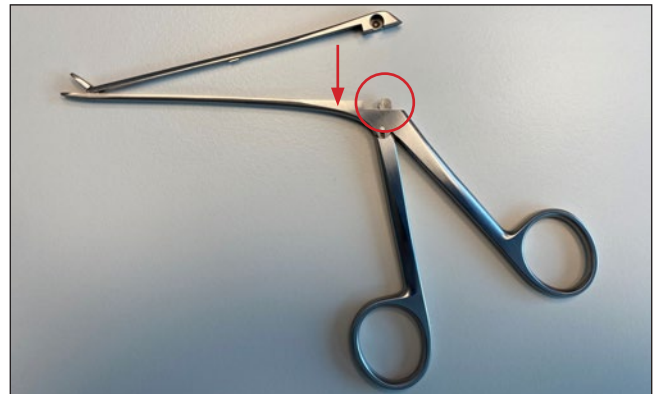

3. The instrument is now completely assembled.

